

ASRD

ADHESIVE SUTURE RETENTION DEVICE

How to Apply ASRD




- 1 Completely clean and dry skin around wound

- 2 Adhere ASRDs 1cm from skin edge; place full thickness suture through device and tighten as necessary to offload wound tension



- 3 Recommended usage is one set of ASRDs for every inch of incisional length

- 4 Choose desired finishing closure technique with sutures, staples, and/or glue

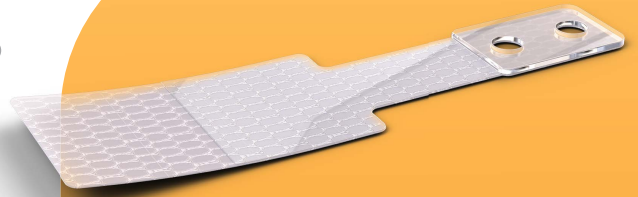

Designed to Prevent Dehiscence

Compared to sutures alone, ASRD supported suture closures resulted in:*



- 80% reduced dehiscence
- 25% increased perfusion
- Earlier weightbearing
- Faster incisional healing
- Reduced skin tearing and blistering
- Reduced undermining, dermal sutures and second intent healing

*Peer-reviewed publications can be found at:
www.suturegard.com/publications




Scan for more information.

info@suturegard.com | suturegard.com

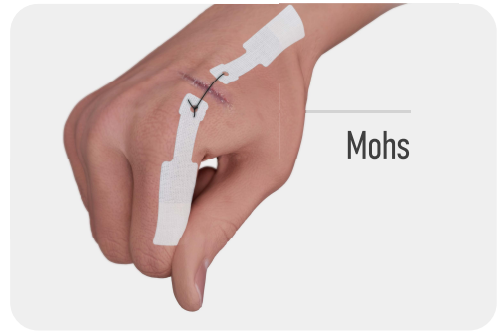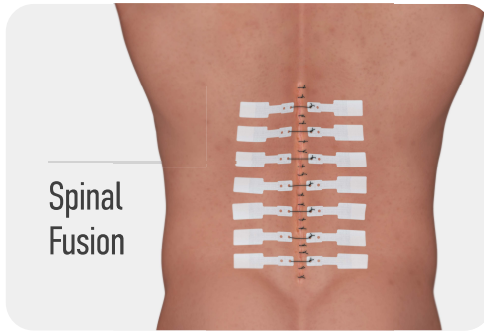


ZONAL[®] Technology

Engineered to offload tension, maintain proper perfusion levels, and minimize wound complications

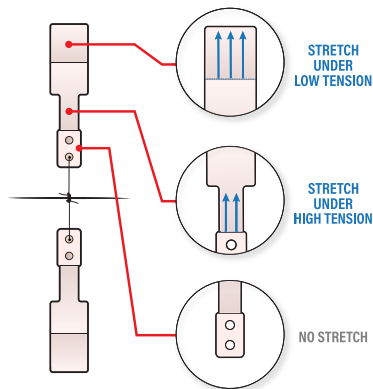


ZONAL[®] Stiffness

Differential stiffness offloads tension to promote optimal healing

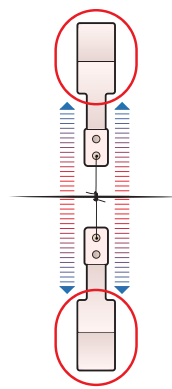

ZONAL[®] Adhesion

Largest surface area furthest from wound

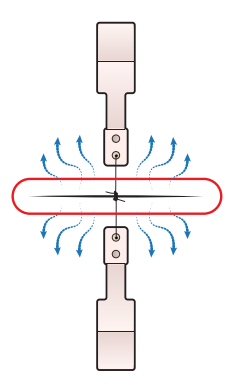

ZONAL[®] Breathability

ASRD spacing maximizes wound breathability