
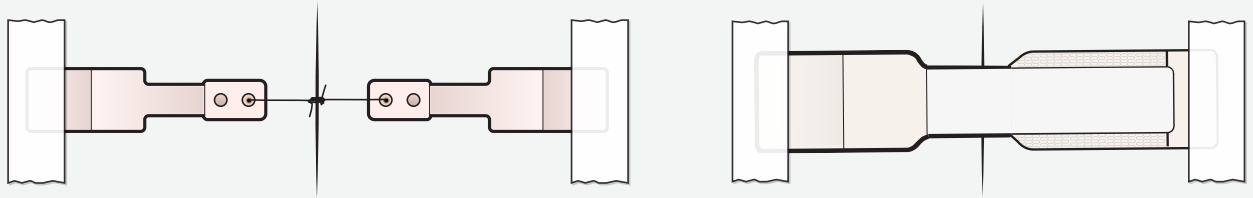


HEMIGARD® Closure Devices

 HEMIGARD® devices are **MUST be kept CLEAN and DRY** in the post-operative period. Moisture will weaken the adhesive and reduce the amount of tension offloading.

 **If the end of the device curls up**, apply an adhesive strip to secure it.





 **DO NOT CUT OR ALTER THE HEMIGARD® DEVICES.**

HEMIGARD® Relieves Tension for Better Healing

Incisions and excisions closed under tension result in thicker scars and increased chance of post-operative re-opening (dehiscence). HEMIGARD® devices relieve tension from the healing wound edge and reduce complications – including dehiscence.

HEMIGARD® ASRD

Offloads stress from a standard nylon suture and allows for closure under very high tension (up to 10 pounds of offloading).

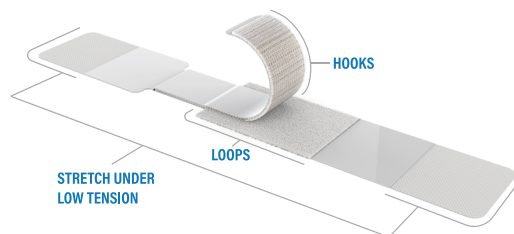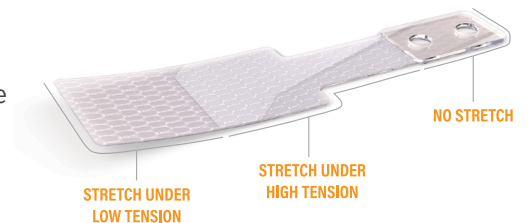

HEMIGARD® INTERLOCK

Uses a strong hook & loop system to relieve wound edge tension without added punctures and is intended for up to 5 pounds of tension offloading.